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Description:

Single piece, threaded endosteal dental implant, including an abutment made of biocompatible, commercially pure grade 4, uncoated titanium.

YAKVIT-1 implant design:

YAKVIT-1 is a self-tapping implant requiring only a small amount of preparatory drilling, allowing for versatility in terms of depth and direction, while at the same time having good primary stability.

Sterility:

The implant is supplied sterilised using gamma ray radiation. It is single use and should only be inserted prior to the expiration date on the label. Do not use the implant if the packaging is damaged or has been opened beforehand. The use of non-sterile implants may cause infectious diseases or tissue infections.

Storage and handling:

The product must be stored in a dry place in the original packaging at room temperature and not exposed to direct sunlight. Incorrect storage may influence device characteristics leading to failure. It is issued in two sterile bags. When it is about to be used, it is recommended that it should be deposited, still sterile, in a sterile container of hydrogen peroxide 10 vol. It should then be picked up using a dissection sterile forceps with serrated jaws and placed in the insertor.

Disclaimer:

This implant should only be used with the components and instruments recommended by the manufacturer. The use of non-recommended products will void any warranty or any other undertaking, either express or implied, given by the manufacturer. The user of the YAKVIT-1 implant is the person responsible for determining if the product is suitable or not for a patient in specific circumstances. The manufacturer declines any responsibility, whether express or implied, and will not be liable for any direct, indirect, punitive damages or damages of any other kind, occurring or in connection with, any errors in judgment or practice of the professional in the use and surgical insertion of the YAKVIT-1 implant.

The clinician is also under a duty to review and understand the application technique to be used for the YAKVIT-1 implant. In case of doubt, the clinician should contact the manufacturer. Since the procedure and the surgical insertion of this product are controlled by the clinician, these will be their responsibility. The manufacturer is not liable for any injury caused as a result.

It is the duty of the clinician to set out a periodic check-up protocol allowing them to address, in the early stages, any possible ensuing peri-implantitis.

Indications:

They can be surgically implanted into the upper and lower jaw bone to fix or support replacement teeth using immediate or delayed loads, for replacements of a single tooth or several, and complete fixed dental arches, for functional and aesthetic oral restoration, in total or partially edentulous patients. For immediate restorations it is necessary to obtain good primary stability and an appropriate occlusal load. If the restoration is immediate two or more adjacent implants should be joined using dental prosthetics. In the event of an immediate restoration in edentulous patients it is necessary to join at least 4 implants.

Dental prosthetic restorations for which the YAKVIT-1 implant can be used are: Individual crowns, bridges and complete dentures which are attached by bonding with the YAKVIT-1 implant.

The Ø 3.5 mm YAKVIT-1 implants are suitable to replace maxillary lateral incisors and/or central or lateral incisors in the jaw.

The Ø 3.5 mm YAKVIT-1 implants are suitable only for single-tooth replacements.

Since the diameter of this implant is so narrow, it is recommended that a prosthetic review be carried out in order to avoid improper loads.

Single piece YAKVIT-1 implants are intended to be used by experienced clinicians, who are capable of placing the implant without losing sight of any subsequent prosthetic restorations.

Recommendations:

It is recommended that the YAKVIT-1 implants be used when there is sufficient occlusal space and it has been decided that the implant may be placed in the correct position to support the cemented prostheses. Should that not be the case, use implants with screw retained abutments.

Contraindications:

The use of dental implants is contraindicated in general, in patients who do not meet the conditions necessary to undergo an oral surgical procedure, those with serious internal diseases, metabolic bone disorders, coagulation disorders, inadequate healing, poor oral hygiene, insufficient maxillary and mandibular growth, generalised poor state of health, uncooperative or unmotivated patients, those suffering from alcoholism or drug addiction, psychotic states, long-term functional disorders resistant to treatment, xerostomia, depressed immune system, diseases requiring the periodic administration of corticosteroids, allergy to titanium, uncontrolled endocrine disorders. Restorations on implants should not bear occlusal overloads.

The Ø 3.5 mm YAKVIT-1 implants are not intended to replace a central incisor, a maxillary premolar or molar canine or a premolar or molar canine in the lower jaw.

The Ø 3.5 mm YAKVIT-1 implants are not intended to be used for multiple replacements.

Qualified contraindications:

Previously irradiated bone, diabetes, haemorrhagic diathesis/anticoagulant treatment, bruxism, parafunctional habits, unfavourable anatomical bone conditions, smoking, uncontrolled periodontitis. Unfavourable inter-maxillary bonding, treatable pathological diseases of the upper or lower jaw, treatable disorders of the oral mucosa, pregnancy, poor oral hygiene. Patients being treated with bisphosphonates.

Local contraindications:

Patients with insufficient bone volume or quality (unless a bone augmentation procedure can be performed), local root fragments. Leftover tissue granulation from previous infections or infections in the adjacent bone. When the appropriate amount of implants required to ensure adequate support cannot be placed.

Side effects or possible complications:

The following side effects associated with the use of dental implants may occur: Lack of osseointegration, loss of bone bonding, dehiscence requiring a bone graft, perforation of the maxillary sinus, injury to the inferior alveolar nerve, persistent pain, dulling, paraesthesia, inflammation and/or infection, excessive bone loss requiring surgery, breakage of the implant, internal bleeding, peri-implantitis, mobility.

Products to which the YAKVIT-1 connects for placement:

The YAKVIT-1 implant must be inserted using:

- An insertor for the mechanical introduction of the implant:** The YAKVIT-1 implant can be inserted using the IKQ3 insertor of the Q-IMPLANT **TRINON** TITANIUM JM system.
- Manual insertion wrench:** The YAKVIT-1 implant can be inserted using manual insertion wrenches IWQS and IWQL from the Q-IMPLANT **TRINON** TITANIUM system.

The remaining necessary instrumentation: Pilot drill, surgical punch and countersink, are standard, and any available on the market can be used.

Special instructions for inserting the YAKVIT-1 implant:

The clinician must be alert and have the necessary concentration required to insert this versatile implant. We recommend that clinicians receive specialised training before using the YAKVIT-1 implant.

1. Inserting the implant: The layout of the different threaded surfaces of the YAKVIT-1 implant allows for its quick insertion.

2. Direction and depth of the implant: The design of the YAKVIT-1 implant allows, during the insertion process:

- Changes of direction for the correct positioning of the abutment over which the prosthesis will be placed.

- Versatility regarding the depth of placement. Thanks to the special arrangement of its cutting edge coils, the implant can be inserted at a greater depth than that determined by the drilling of the osteotomy, leading to the correct occlusal height of the abutment from very beginning. The implant should be inserted into the bone until the last coil is at the same height as the bone cortex. However, should clinical conditions require it, it is possible to insert it at a depth of 1 mm higher or lower.
3. Degree of tightening: With mechanical inserters, a torque higher than 32 Nw is not recommended, so as not to damage the inserting or contra angled wrenches. If insertion is completed using a hand wrench, a higher torque may be used, depending on the clinician’s experience.

Important: Read carefully

Surgical procedures:

Bone quality must be taken into account when drilling. (See the *TABLE “Recommended drilling sequences taking into account bone quality”* to ensure optimum primary stability by applying immediate function). Drilling must be carried out with abundant and constant external sterile saline irrigation.

TABLE “Recommended drilling sequences taken into account bone quality”

IMPLANT DIAMETER YAKVIT-1		Soft bone type IV	Medium bone Type II-III	Dense bone Type I
Ø 3.5	Step 1	Round drill Ø 2	Round drill Ø 2	Round drill Ø 2
	Step 2	Pilot drill Ø 1.7	Pilot drill Ø 1.7	Pilot drill Ø 1.7
	Step 3		Drill Ø 2.4	Drill Ø 2.4
				Drill Ø 3.5 (*)
Ø 4.5	Step 1	Round drill Ø 2	Round drill Ø 2	Round drill Ø 2
	Step 2	Pilot drill Ø 1.7	Pilot drill Ø 1.7	Pilot drill Ø 1.7
	Step 3	Dril Ø 2.4	Drill Ø 2.4	Drill Ø 2.4
	Step 4		Drill Ø 3.5	Drill Ø 3.5
	Step 5			Drill Ø 4
	Step 6			Cortical countersink

Note: All measurements are expressed in mm.

(*): *Insert the Ø 3.5 mm drill, drilling to half depth. Following that, you can attempt to insert the Ø 3.5mm implant.*

You must be aware that the drill is up to 1 mm longer than the inserted implant. Please take into account this extra length when drilling close to vital areas.

Positioning the implant:

The implant is the most important part of the restoration. It provides the basis for planning the surgical technique to be used. Communication between the patient, the dentist, the surgeon and the dental technician is essential to achieving the desired prosthodontic outcome.

To determine the topographical layout, the orientation of the axis and the implants to be selected, we recommend proceeding as follows:

- Use a wax mould/cast over the pre-prepared study model.
- Define the superstructure type.

The wax mould/cast can then be used as the basis for a radiographic template or individualised drilling, and for a temporary cap.

The abutments must always be loaded axially. Ideally, the longitudinal axis of the implant will be placed using the cusps of the opposing tooth. The cusps of the replacement not should be excessively etched since this can give rise to non-physiological loads.

The diameter of the implant, the length, type, position and number are factors that must be determined on an individual basis taking into account the patient’s anatomy and spatial conditions (e.g. poorly positioned or crooked teeth). The measurements set out here should be taken as indicative minimum values

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